

Sanislink Node Calibration Guide

Welcome to the User Sovereign Health Infrastructure. To ensure your hardware node captures encrypted biometric data accurately, a perfect fit is mandatory.

CRITICAL: Print this page at **100% SCALE**, and set all page margins to **0**. Do not select "Fit to Page" or "Scale to Fit" in your printer settings.

STEP 1: VERIFY PRINT SCALE

Place a standard credit card inside this box. If the card does not fit the outline *perfectly*, your print scale is wrong.

STOP HERE and reprint at 100%.

If it fits, proceed to Step 2.

STEP 2: THE NODE STRIP (Most Accurate)

- ① Cut out the Node Strip along the dotted lines.
- ② Wrap the strip around the base of your intended finger. Stagger the strip as little as possible after overlapping it up and down, then use scale line 0 (the WHITE line) to find within white scale lines from 7 to 13.
- ③ The scale line number that lines up with 0 is your Sanislink Node Size. If 0 lines up between two sizes, we recommend choosing the larger size.

STEP 3: THE LEGACY MATCH (Alternative)

If you already have a ring that fits perfectly on the finger you intend to use for Sanislink:

- ① Place your existing ring over the circles below.
 - ② Match the **INSIDE** edge of your ring with the drawn circle.
- If the ring is between two sizes, we recommend choosing the larger size to accommodate daily finger swelling.

International/ American Code	Diameter (mm)	Finger Circumference (mm)	Weight* (kg)
No.7	17.2	54.0	45-55
No.8	18.1	56.9	55-60
No.9	18.9	59.5	65-70
No.10	19.8	62.1	70-75
No.11	20.6	64.6	75-80
No.12	21.4	67.2	85-90
No.13	22.2	69.75	95-100


* Body weight is for reference only.




Pro Tip: Ensure the paper can slide comfortably over your knuckle. (Use tape or glue to help secure the strip.)




No.

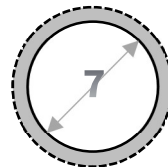


8

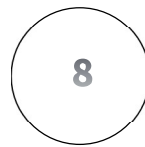

9

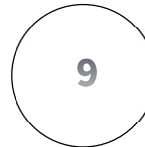

10

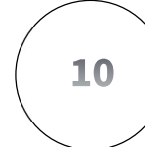


11



12



13

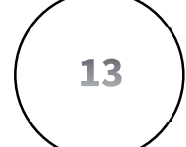

⚠ THE GOLDEN RULES OF SIZING & DISCLAIMER:

Your fingers swell during sleep and exercise. Measure at the end of the day when your fingers are largest. The Sanislink ring houses military-grade sensors. It must sit flush against your skin but should not cut off circulation.

LEGAL NOTICE: By submitting your size, you confirm it has been verified using this calibration guide. Sanislink rings cannot be resized. Incorrect sizing submissions that require an exchange will incur **round-trip shipping costs** and a **\$15 restocking fee** by consumer.